

## KRUUSE Fun-Flex Care, Inside-Out

Some nurses and vets prefer to bandage inside-out. On this Fun-Flex Care version the pattern is printed on the inside of the bandage which leaves it visible once the bandage is applied.

Send your patients home with a caring message. Wish them good luck, tell them they are a star or find your own special messages in the 5 patterns.



### One way or the other

- Which way do you prefer to bandage?
- The print on the bandage determines which way you apply it on the patient
- KRUUSE Fun-Flex Care comes in both versions

*Simply choose your direction!*



Scan QR code  
to watch  
the video



# KRUUSE Fun-Flex Care, Inside-Out

Superior quality flexible cohesive bandage for fixation of dressings and controlled support of sprains and dislocations. Strong, comfortable, and user-friendly. Easy to tear.

## Recommended use for KRUUSE Fun-Flex care, Inside-Out

- Controlled support and compression of joints, sprains, dislocations
- Secures dressings, splints, IV catheters, etc.

## Instructions for use

### Application

- Fixate the end of the bandage and apply evenly over a larger area of the injured body part stretching the bandage according to the level of compression needed
- When application is done, do a straight tear of the bandage. Do not twist, as it will make the tearing more difficult
- To finish off, gently squeeze the bandage layers together to enhance cohesiveness. Use KRUUSE Fun-Flex as secondary/tertiary bandage on top of primary/secondary dressings

### Removal

- Unroll the bandage or cut gently with bandage scissors. Be careful not to cut into the skin

### Do not use

- If patient or user is allergic to latex

*For veterinary use only*

Cat. No	Description	Size, Metric	Size, Imperial	Package
160715	KRUUSE Fun-Flex Care, Inside-Out	5 cm x 4,5 m	2" x 15 ft	10/pk, 2 of each pattern
160716	KRUUSE Fun-Flex Care, Inside-Out	7,5 cm x 4,5 m	3" x 15 ft	10/pk, 2 of each pattern
160717	KRUUSE Fun-Flex Care, Inside-Out	10 cm x 4,5 m	4" x 15 ft	10/pk, 2 of each pattern



## Let's wrap it up!

### Reliable and stable

The high weight of the material, 35 g/m<sup>2</sup>, makes the bandage strong and firm, yet comfortable. Provides excellent compression.

### Easy to tear

No scissors needed! Simply pull in opposite directions to tear in a straight line.

### Skin-friendly

KRUUSE Fun-Flex is coated with latex, which makes the bandage cohesive. It doesn't stick to surfaces like skin or fur and the porosity makes it breathable.

### Very flexible

KRUUSE Fun-Flex can be stretched to almost double the unrolled length. You can bandage all types of injuries on the patient's body and still control the compression.

### Excellent elasticity

Spandex threads provide suitable shrinkage, which is necessary to give firm compression and fixation. With proper application the bandage will not constrict.

### Easy to remove

The features of KRUUSE Fun-Flex cohesive bandages make them very easy to remove even without scissors.

### Hygienic

The bandage is water-resistant, which helps prevent water from penetrating the layers. Each roll is individually wrapped in an easy-to-tear pouch.

### Good patient and user compliance


The fact that the bandage is very light, does not constrict and is not sticky to touch makes it more comfortable for the patient and easy to work with for the nurse, vet, and owner.



Easy to tear



Latex threads

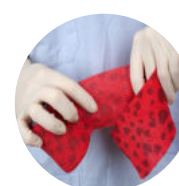

Spandex threads